



W H A L T I M A

MASSAGING MAT



MODEL: FIJI

USER MANUAL



Advances in science and technology have significantly changed our lives, and health care is becoming a full-fledged movement toward a healthy lifestyle.

The FIJI massage chair actually plays the role of a massager, providing maximum relaxation for the neck, shoulders, back, waist and buttocks anytime and anywhere. Its S-shape perfectly mimics the curves of the body, and the adjustable arm position fits users of different heights. The design provides heating for the neck, shoulders and back to help achieve even deeper relaxation. The air jet massage also relaxes the waist and buttocks, reducing pressure on the spine and strengthening muscles. This stylish and elegant massage chair fits perfectly into any chair to suit your taste. The back strap is used to secure the device to prevent damage.

Thank you for purchasing this device. Please read this user manual carefully before using it to ensure the right installation and operation of the device. Keep this user manual for future use.





CONTENTS

Precautions	1
Names and functions of all components	3
External structure	3
Internal structure	3
Control panel	4
instructions for the control panel	5
User manual.....	7
Operation manual	7
Cleaning and maintenance	
8 Problem solving	
9 Technical specifications	
9	


PRECAUTIONS

Important safety information:


- ▶ Before using, read this manual carefully.
- ▶ The following precautions will help ensure proper operation of this equipment. The types of potential accidents caused by improper use are divided into two points: "WARNING" and "CAUTION".
- ▶ Read all sections marked "WARNING" and "CAUTION" and follow all instructions from this manual.

 WARNING!	Improper operation may lead to injury to the the user.
 CAUTION!	Improper use may lead to injury to users or damage to the device.
 DO NOT USE!	 DO NOT DISMANTLE!


This device is not suitable for people listed below. We recommend that you consult your doctor before using the device.

 WARNING!	<p>This device is not suitable for people:</p> <ul style="list-style-type: none">▶ with implants in the body,▶ with malignant tumors, acute diseases, heart disease or severe hypertension as well as pregnant women,▶ with osteoporosis, joint deformities or recent injuries,▶ children and people with disabilities are allowed to use only under close supervision,▶ with serious skin diseases or skin damage,▶ This appliance is equipped with a heating function. Please observe safety precautions when using the control panel.
-------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

Precautions for using the control panel


 WARNING!	<ul style="list-style-type: none">▶ Do not touch the control panel with wet hands or pull on the cable.▶ Avoid contact with liquids.▶ Do not place heavy objects on the control panel.
-------------------------------------------------------------------------------------------------	------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

Recommended conditions for use





 WARNING!	<ul style="list-style-type: none">▶ Do not use this device at temperatures above 40°C.▶ Do not use this device in a humid or dusty environment.▶ Unplug the device from the socket during a thunderstorm.▶ Do not use the device outdoors.▶ Place the device on a level, smooth surface.▶ Do not store the device near heat sources, open flame or in direct sunlight.
-------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

PRECAUTIONS

Warnings regarding power outlets and power cord.


 CAUTION!	<ul style="list-style-type: none"> ▶ Make sure that the power supply voltage indicated on the nameplate matches the parameters of your electric power source. ▶ Do not disconnect the power cord with wet hands to avoid electric shock. ▶ Do not pull on the cord, grab the front plastic end to disconnect it. ▶ Do not pinch the AC cord with a device or other heavy objects. ▶ Do not damage, bend, pull or twist the power cord. ▶ Use only a suitable power adapter, consult your supplier or a qualified specialist to replace it. ▶ If the power cord is damaged, replace it by contacting the manufacturer, service or any qualified professional to avoid danger.
------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

Recommendations for safe use.


 WARNING!	<ul style="list-style-type: none"> ▶ Do not place heavy objects on the device. ▶ Do not use other therapeutic devices simultaneously with this device. ▶ Use this device only for its intended purpose as described in the user manual. ▶ The device should only be used by one person, do not use it by two or more people at the same time to avoid damaging the device. ▶ The device is intended only for users weighing less than 90 kg. ▶ Do not use the device directly after a meal, it may cause discomfort. If you experience discomfort while using the device, stop using it immediately. ▶ It is not recommended to use the device for more than 15 minutes. Check the upholstery and other components, if there is any damage, do not use the device and contact a qualified professional to carry out maintenance. ▶ Maintain correct posture while using the device. ▶ If the power is turned off during use, press the power button to turn off the device and avoid hazard that may occur when the power is turned back on.
 FORBIDDEN!	<ul style="list-style-type: none"> ▶ Do not use the device if it has damaged/open parts. ▶ Do not put or place heavy objects on the device. ▶ Watch that children do not play with the massager. ▶ Do not fall asleep while using the device. ▶ Avoid using the device as intoxicated by alcohol. ▶ Do not put your hands or foreign objects into the openings of the device to avoid injury to the user or damage to the device. ▶ If there is smoke or unusual odor while using the device, stop using the device immediately.
 DO NOT DISMANTLE	<ul style="list-style-type: none"> ▶ Do not dismantle the device yourself to avoid damage. ▶ Do not attempt to repair or disassemble the unit without proper authorization. Contact an authorized dealer or service agent.
 DISCONNECT THE POWER CORD	<ul style="list-style-type: none"> ▶ Never plug the device into a socket or pull out the power cord with wet hands to avoid electric shock or injury. ▶ Unplug the power cord from the extension cord after use. ▶ Remove the plug from the socket after turning off the power.

NAMES AND FUNCTIONS OF ALL COMPONENTS

External structure

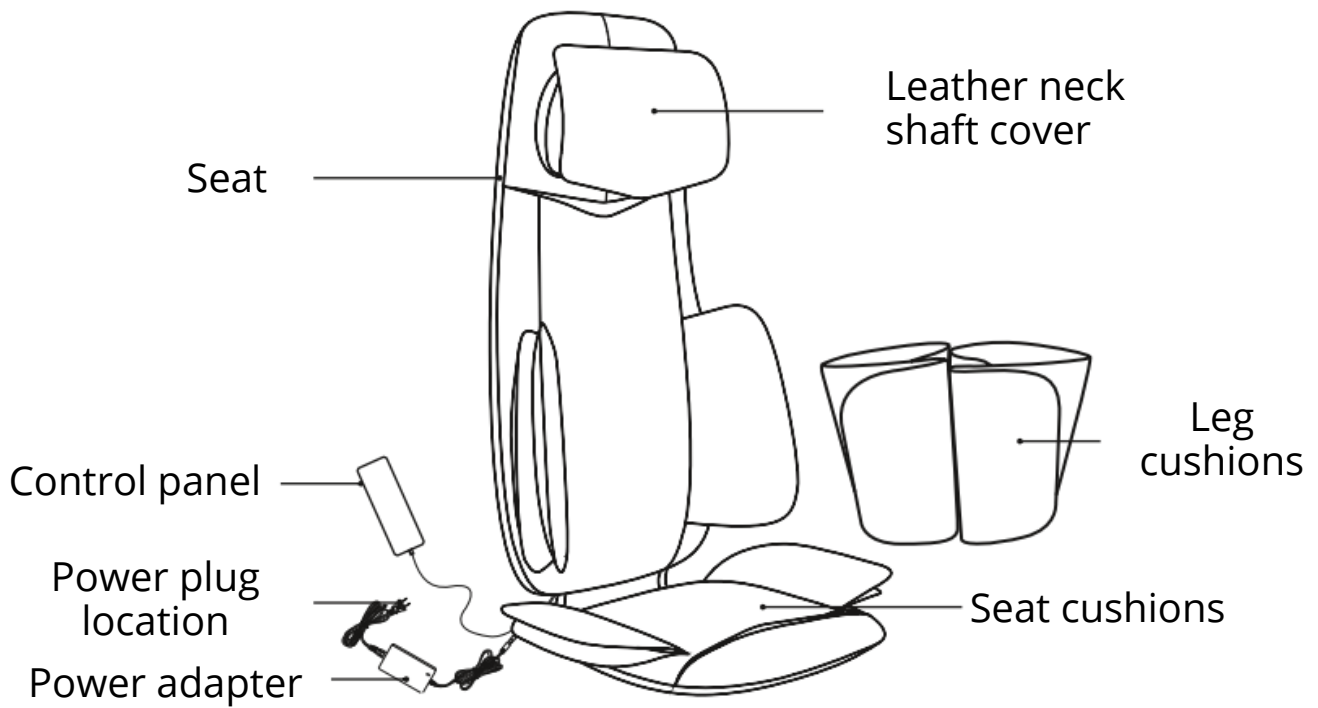

Internal structure

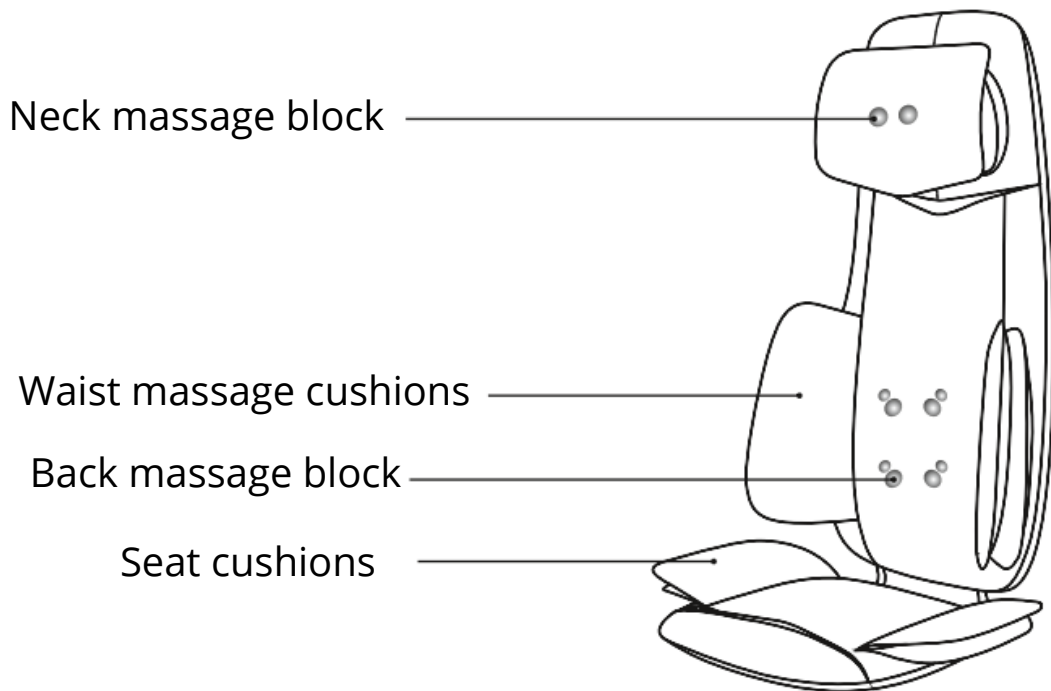

NAMES AND FUNCTIONS OF ALL COMPONENTS

Control panel



WARNING!

- ▶ This massager is equipped with a heating function, this should be noted for those with reduced heat sensitivity.

NAMES AND FUNCTIONS OF ALL COMPONENTS

Control panel operating instructions




Power on/off button:

Press this button to turn on and start the massager. The timer set for 15 minutes will be activated (the power indicator will light up). Press the button again to turn off the massager. The massage mechanism will return to the initial position (the indicator flashes and goes off when the restoration of the initial position is complete).




After starting the massager:

When the massager starts, the neck massage will start clockwise in the default mode; press this button to switch to the counter-clockwise rubbing mode; press again to turn off the neck massage; the neck heating function will be turned off if the neck rubbing massage is interrupted.

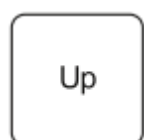

Button for lifting the mechanism for upward neck massage:

Press and hold this button (the indicator lights continuously) to raise the neck massage mechanism when connected to a power source; release the button to stop (the indicator goes out); the indicator flickers when the mechanism reaches the extreme position.

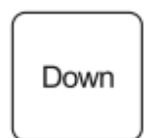

Button to lower the neck massaging mechanism down:

When connected to a power source, press and hold this button (the indicator lights steadily) to lower the neck massaging mechanism down; release the button to stop it (the indicator goes out); the indicator flashes when reaching the extreme position.



Automatic neck massage button:

If the device is in neck massage mode, press this button to start/stop the automatic full neck roll function.



Neck warming button:

Press this button when you perform a neck massage to turn the neck heating function on/off.



When the massager is turned on:

When the massager is turned on, the full back massage starts by default. Press this button to switch to lower back massage, press it again to switch to upper back massage, and a third time to turn off the back massage. The back heating function will be turned off when the back massage stops.



Button to turn back heating on:

After starting the back massage/spot back massage, press this button to turn the back heating function on/off.

NAMES AND FUNCTIONS OF ALL COMPONENTS

Back
Spot

Spot back massage button:

Pressing this button will focus the massage on the current position, pressing it again will stop the spot massage mode. In addition, you can press the back heating button to turn on the heating.

Air
Pressure

Air massage button:

When connected to a power source, pressing this button will turn on the air massage function. When the leg cushions are attached to the massager, the lumbar air massage mode is the default setting (body indicator constantly lit); pressing this button will switch the mode to leg air massage (leg indicator constantly lit). Pressing this button again will switch to full air massage mode: (lumbar + legs). Pressing this button for a third time will turn off all air massage functions (air massage indicator will go out).

If the leg cushions are not attached to the massager, the lumbar air massage mode is the default setting (the body indicator constantly lights up and the air massage indicator flashes); pressing this button will switch to the medium intensity air massage mode (the body indicator constantly lights up and the air massage indicator flashes quickly); pressing this button for a third time will turn off the air massage function (the air massage indicator goes out).

Back
Intensity

Back heating intensity button:

When performing a back massage/spot back massage, medium intensity is the default setting. Pressing this button will switch to high intensity, and pressing the button again will switch to low intensity.

Auto

Automatic mode button:

When connected to a power source, pressing this button will enable/disable the automatic massage mode. When the automatic massage mode is on, you can press another button to exit the automatic mode and adjust your own massage mode.




WARNING!

- ▶ Do not touch the remote control with wet hands to avoid electric shock or damage to the control panel.
- ▶ To avoid abnormalities, make sure no water or other liquids get on the control panel.

USER MANUAL

User manual

- ▶ Check the device for damages before use.
- ▶ Place the device on a flat surface (e.g. on a sofa or armchair), secure it with the belt on the back.
- ▶ Connect the power plug to the power supply and the power switch to the electrical socket (100-240 V) (see figure below).
- ▶ Press the power button to start the massaging device, provided that it is connected to a power source.
- ▶ Always unplug the device from the socket immediately after use.

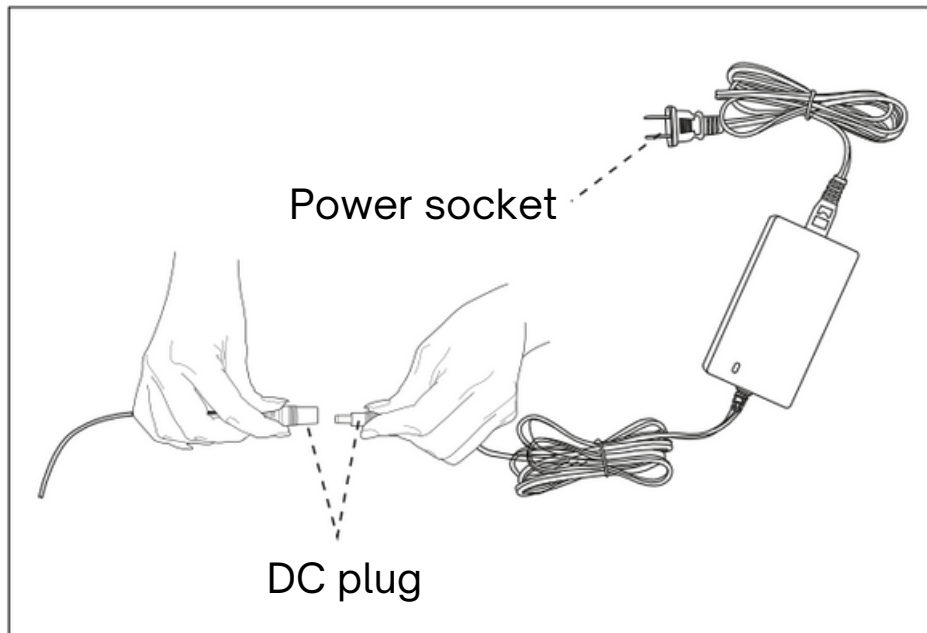

WARNING!

- ▶ By default, the device runs for 15 minutes, after which it automatically shuts down. Press the power button to restart the device.
- ▶ During use, the back of the massage chair should remain in an upright position, do not use the device while lying down.
- ▶ Do not place your feet on the device.
- ▶ Do not carry this device by the power cord or use the cord as a handle to avoid damaging it.
- ▶ Turn off the power and store the massager when not in use to avoid its damage.
- ▶ Turn off the power and put the massager away when not in use so that children do not accidentally turn it on.

CLEANING AND MAINTENANCE

User manual

Cleaning

- ▶ Before cleaning, make sure to turn the power off and unplug the power cord from the socket.
- ▶ Use a soft cloth to clean the surface of the device.
- ▶ To remove stains, wipe them with a soft cloth soaked in water or a mild pH-neutral cleaner, then wipe it dry with another piece of soft cloth.



WARNING!

- ▶ Never plug or unplug the power cord with wet hands.
- ▶ Avoid contact with liquids.
- ▶ Do not use gasoline, thinners or other solvents for cleaning the device.
- ▶ Do not try to disassemble the device on your own when it is wet, let it dry.

Storage methods

- ▶ Always turn off the power before storing the device.
- ▶ Store the device in an environment free of moisture and volatile gases, and do not store it at high temperatures.
- ▶ If the device will not be used for a long time, it is recommended to pack it in a box to protect it from dust.



WARNING!

- ▶ Do not leave the device directly in the sunlight for an extended period of time. This may cause fading or damage.

Advice on the heating system

With daily use, check the device for the following signs:

- ▶ burnt smell;
- ▶ flashing power when touching the charging cable;
- ▶ hot power cord;
- ▶ defective power cord;
- ▶ other abnormal symptoms.



WARNING!

- ▶ Please stop using the device to avoid damage or accidents if you observe any of the abovementioned symptoms.
- ▶ Turn off the power with the button and remove the plug from the socket, then contact your local distributors or supplier for repairs.
- ▶ Do not attempt to repair or disassemble the device yourself.

PROBLEM SOLVING

Nr	Fault symptoms	Possible causes	Possible solutions
1	Noise is heard during use	This sound is normal, its caused by the friction of the massager against the fabric covering.	No action is required.
2	Massaging device does not start	The power adapter is connected incorrectly	Connect the power adapter properly
		The power switch is in the off position	Turn on the power switch
		The power cord is not plugged into the socket	Plug the device into a socket

If the problem persists, contact the dealer for service at the phone number listed on the purchase contract.

TECHNICAL SPECIFICATIONS

TECHNICAL SPECIFICATIONS:

Name of the device: Massaging chair

Brand: WHALTIMA

Model: FJI

Rated voltage: 210-230 V

Input voltage: 12 V

Frequency: 50 Hz

Rated power: 60 W

Protection class (IP): IP5

Manufactured in China

Producer: SHANGHAI RONGTAI HEALTH TECHNOLOGY CORPORATION LIMITED.

Adress: Nr. 146, Mali West New Village, Guanhu Street, Guanlan, Shenzhen, China.

Don't throw this manual away.

The appearance of the product may not match the representation. This manual has undergone careful editing and verification, but this does not exclude the possibility of errors during creation and printing. We apologize for any misunderstandings in advance. The right of final interpretation belongs to the Company. This manual is a literal translation and reflects the information provided by the manufacturer.



Improper disposal of waste can harm the environment.



W H A L T I M A